

Bodypod®



Simplifying removals with the all new Bodypod

GUIDELINE | STEP BY STEP

Proprietary notice

The information disclosed in this manual is the property of Tetcon Global BV, Eindhoven, The Netherlands. Tetcon Global BV reserves all patent rights, proprietary design rights, manufacturing rights, reproduction use rights and sales use rights thereto, and to any article disclosed therein except to the extent those rights are expressly granted to others or where not applicable to vendor proprietary parts.

User information

Bodypod Transfer Mattress comes with a comprehensive 'User Manual', for the correct usage protocol. Tetcon Global BV cannot be held responsible for equipment that has not been maintained in accordance with its guidelines, specification or for the incorrect use of equipment under working conditions.

Company information

Tetcon Global BV
Johannes Buijslaan 20
5652 NK Eindhoven
The Netherlands
www.s-capeplus.com/en/
www.tetcon-ge.com



What will you learn?

In this eBook, you will learn the basic techniques that are required to operate the Bodypod Transfer Mattress.

1. The safe use

In this eBook, each user step is carefully described to ensure the safe usage.

2. Teach your colleagues

By closely observing the steps in this eBook, you will be able to teach your colleagues the safe usage.

3. Avoiding mistakes

It is important to avoid user mistakes. Therefore, it is recommended to reread each step before taking action.

Bodypod

The Bodypod Transfer Mattress has been designed to provide a dignified body removal when a stretcher cannot be used.

Rigid removal stretchers or trolleys may not always be suitable for a house removal especially when you are dealing with narrow corridors or rotating stairways and trying to negotiate tight corners.

The flexible lightweight Bodypod Transfer Mattress has been designed and manufactured to assist in these situations.

This step-by-step guide allows you to practice where and whenever you want.

See how easy it is, then try it yourself.

Bodypod[®]

Bodypod®



Learn the safe use of the Bodypod Transfer Mattress.
Scroll down and let's get started.





Storage bag

4 positioning straps

STEP 1

Remove the Bodypod from the storage bag. Open up the flaps and release the positioning straps.



STEP 2

Position the deceased in the centre of the Bodypod .



Metal rings

STEP 3

Tie the positioning straps around torso and around the legs of the deceased.
Tighten the straps by pulling them through the metal rings on the side of the Bodypod.
Fix the straps with the hook and loop fasteners to stabilise the body.



Head strap

STEP 4

Place the head strap around the forehead of the deceased and press it against the reserved black area.



STEP 5

Now pull the long flaps vigorously towards each other.



STEP 6

Now pull the long flaps vigorously towards each other. Connect and stick the hook and loop fasteners. Do this as tight as possible.



STEP 7

Connect and close the flaps at the feet and head end with the rest of Bodypod. Make sure this is tight and secure.



STEP 8

Use the pulling straps at the feet and head end to move the deceased.



STEP 9

Slowly descend the stairs. Descend a few steps yourself and pull the Bodypod towards you. Make sure you hold the banister and keep contact with the Bodypod using your leg.



STEP 10

Ensure that a second and/or third person is present to firmly hold the strap on the head end.

This ensures braking and steering while descending the stairs. Descend at a slow pace.

Communicate with your colleague and go down step by step.



STEP 11

Once arrived downstairs - use the handles on the side to transfer the Bodypod to the brancard or stretcher.



STEP 12

Once arrived downstairs - use the handles on the side to transfer the Bodypod to the stretcher or trolley.



STEP 13

For transport, you can also use an additional concealment body cover.

IMPORTANT

If serious damage to your Bodypod is observed, do not throw away the entire system.
You can replace the Bodypod cover using the zipper on top of the head section.



SAFETY INFORMATION

Never overestimate yourself! A transfer via a staircase can be challenging! If in doubt as to whether you are able to transfer a deceased down a staircase yourself, you must always leave the task to someone else. Your own safety is of the utmost importance! Regularly practicing using the Bodypod during exercises and always observe the instructions. These instructions contain information concerning the safe use of the Bodypod. To ensure that the Bodypod is used correctly and safely, this information must be read prior to placement and usage of the Bodypod.

- Always observe the instructions given in the instruction manual when using the Bodypod.
- The use of alcohol and/or prescription and over-the-counter medicine may have a negative effect on the skill of the person who operates the Bodypod.
- Do not remove any parts of the Bodypod.
- This Bodypod is not intended for recreational use.
- Do not use flammable vapours, materials or liquids, such as petrol, alcohol, etc., close to the Bodypod.
- Not observing the instructions given in the instruction manual provided with the Bodypod may result in serious physical injury.
- Sharp edges or protrusions may damage the Bodypod and, therefore, reduce the effectiveness of the product.
- Practice using the Bodypod in order to guarantee its correct and safe use in the event of a transfer.
- The Bodypod must be carefully checked for damage after it has been used. If serious damage (e.g., holes bigger than 2 cm) is observed, the Bodypod must not be reused.
- The Bodypod has a 1-year warranty for material and manufacture faults, which is valid from the date of purchase. The manufacturer may ask you for proof of the purchase date. Therefore, it is recommended to keep the invoice in a safe place. Faults or problems with the use of the Bodypod as a result of an accident, misuse, incorrect use, alterations, vandalism, incorrect placement, incorrect maintenance or administration, as well as overdue normal and routine maintenance, are not covered by the warranty.
- The manufacturer and distributor is not liable for (consequential) damage as a result of the use of the Bodypod

The logo for Bodypod, featuring the word "Bodypod" in a bold, black, sans-serif font with a registered trademark symbol (®) to the upper right of the letter 'd'. The logo is centered within a white rectangular background.

**TUTORIAL & EBOOK
COMPLETE**

Bodypod[®]